NL

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0481

EN

## Instructions for use HeltiQ Honey Wound Gel

HeltiQ Honey Wound Gel contains: 40% medical grade honey, hypoallergenic lanolin, propylene glycol, PEG 4000 and vitamins C & E. When HeltiQ Honey Wound Gel is in contact with the wound, fluid is drawn from the surrounding tissues. This creates a moist wound healing environment which stimulates the wound healing process.

### HELTIQ HONEY WOUND GEL:

- Has antibacterial properties.
- Dilutes gradually and can be removed easily.
- Prevents adhesion to the wound bed when covered with a dressing.
- Due to the use of natural ingredients, each batch may vary in colour (yellow brown), viscosity and may contain visible sugar crystals. These variances will not affect the efficacy and use of the product.

### INDICATIONS

Superficial and acute wounds e.g. cuts, abrasions, surgical wounds, donor sites from skin grafts, and superficial 1st & 2nd degree burns. Chronic\* wounds e.g. bedsores, leg ulcers and diabetic foot ulcers. Wounds with microorganisms. Wounds caused by a tumour or metastasis or the treatment thereof. Helps to deodorise and remove dead tissue.

\* *Whether it is a chronic wound should be determined by a physician. Use this product for its intended purpose after consulting the physician.*

### CONTRAINDICATIONS AND CONSTRAINTS

No contraindications are known to date. Do not use on patients who are sensitive to the product or any of its components.

### PREVENTIVE MEASURES AND COMMENTS

1. Always consult a doctor for wound management advice.
2. Do not use new products that are damaged or expired.
3. Due to dissolving dead tissue, the wound may appear to increase in size after the first dressing changes. This is normal and occurs as the dead tissue is removed, preparing the wound bed for healing.
4. In the event of intolerance to HeltiQ Honey Wound Gel or one of its components, remove the product and carefully clean the wound area.
5. HeltiQ Honey Wound Gel is for single patient use, the product may only be re-used with the same patient.
6. Always re-seal the product after use.
7. Use the product within 3 months after opening.
8. Dispose of the product with normal household waste.

EN

## Instructions for use HeltiQ Honey Wound Gel

### INSTRUCTIONS FOR USE

1. Remove the lid, clean the mouth of the tube and the top of the lid with an alcohol swab.
2. Push the top of the lid into the foil seal that covers the nozzle, do not squeeze the tube at this time.

### Application to the wound

1. Apply a thin layer of HeltiQ Honey Wound Gel to the wound and surrounding tissue.
2. Cover the wound with a wound plaster, non-adherent wound compress or wound dressing and fixate with an conforming bandage. Secure this with an adhesive plaster.
3. Inspect the wound and the surrounding tissue regularly.

### FREQUENCY OF USE

This depends upon the amount of wound fluid and the needs of the wound. Re-apply the product when it has dissolved ( $\pm$  once every 24-48 hours).

### STORAGE PRECAUTIONS

Store at room temperature between 5 °C and 25 °C (41-77F). Avoid direct sunlight and moisture. HeltiQ Honey Wound Gel is a natural product and may become liquid when exposed to higher temperatures. If this is not desired, place the product in a refrigerator prior to use.

 Theo Manufacturing B.V.  
Sleperweg 44, 6222 NK Maastricht, Nederland

 Koninklijke Utermöhlen NV  
De Overweg 1, 8471 ZA Wolvega, Nederland  
[www.heltiq.nl](http://www.heltiq.nl)

This product is a medical device.